

ALCOHOL WIPES FOR RAPID SURFACE DISINFECTION

Incides® N

Ready-to-use wipes for disinfection of medical equipment and general surfaces

The fast acting ready-to-use alcohol based disinfectant wipes combine short contact times with a broad efficacy spectrum and easy handling.

Available in canister and refill format, Incides N wipes also provide:

- ▲ Short contact times
- ▲ Direct use
- ▲ Dermatological approval
- ▲ Quick drying without leaving critical residues



Incides® N

Alcohol based wipes for disinfection of medical equipment and general surfaces

Why Use Incides® N?

BROAD EFFICACY SPECTRUM IN SHORT CONTACT TIMES

With its broad efficacy against micro-organisms including bacteria, TB, yeasts and various viruses in short contact times, the alcohol based Incides N wipes offer excellent disinfection properties on all alcohol-resistant surfaces. They are ideal for intensive care area and other high risk areas.

PERFORMANCE ASSURED FOR RELEASED LIQUID

The pre-impregnated wipes are performance proven for solution release under real working conditions ensuring complete efficacy is achieved on your surfaces disinfected with Incides N.

RELIABLE AND CONVENIENT

The ready-to use pre-impregnated wipes reduce preparation and dosing efforts and combine rapid disinfection with high material compatibility, ensuring efficient and reliable product use.



Incides N
Refill 5 x 90 wipes

Incides N
Canister 6 x 90 wipes

MICROBIOLOGICAL AND VIROLOGICAL TEST RESULTS

EFFICACY (PROVEN WITH RESPECT TO THE SOLUTION RELEASED FROM THE WIPE)	CONTACT TIME
Disinfection of surfaces acc. to VAH with mechanical action (bactericidal, yeasticidal, tuberculocidal) (clean & dirty conditions)	5 min
Rapid disinfection of surfaces acc. to DGHM (German Society for Hygiene and Microbiology) method 2001 with mechanical action (bactericidal, yeasticidal, tuberculocidal)	1 min
EN 13727 (bactericidal) (clean & dirty conditions)	1 min
EN 13624 (yeasticidal) (clean & dirty conditions)	1 min
EN 14348 (tuberculocidal, mycobactericidal) (clean & dirty conditions)	1 min
Effective against enveloped viruses (incl. HIV, HBV, HCV) acc. to RKI recommendation 01/2004*	30 sec
Adeno*	15 min
Papova*	15 min
Rota*	5 min

* DVV 2008

EUROPEAN HEADQUARTERS: ECOLAB EUROPE GMBH

Richtistrasse 7
8304 Wallisellen
Switzerland
+41 (0) 44 877 2001
www.ecolab.com

ECOLAB
DEUTSCHLAND GmbH
Ecolab-Allee 1
40789 Monheim am Rhein
Germany
+49 (0) 2173 599 0
www.de.ecolab.eu

CE 0297

QUICK.
FLEXIBLE.
RELIABLE.

APPLICATION AREA

Incides N is a ready-to-use wipe for disinfection of alcohol-resistant medical devices and general surfaces in healthcare settings.

INSTRUCTIONS FOR USE

Take single wipe. Wipe surface, let it dry and do not rinse. Complete wetting of all surfaces to be disinfected is important. Close canister after use.

INGREDIENTS

100g of soaking solution contains the following active ingredients: 35g 2-Propanol, 25g 1-Propanol.

PACKAGING AND ORDERING INFORMATION

PRODUCT	OUTER SIZE	ORDER CODE
Incides N Canister	6 x 90 Wipes	XXXXXX
Incides N Refill pack	5 x 90 Wipes	XXXXXX

STORAGE

Product should be stored in the original container below 25°C in a cool, dry location out of direct sunlight.

USE BIOCIDES SAFELY. ALWAYS READ THE LABEL AND PRODUCT INFORMATION BEFORE USE. FOR PROFESSIONAL USE ONLY.

ECOLAB®